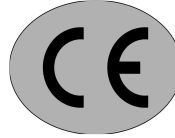
PRESSOJET

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SUMMARY :

	Page
<i>INTRODUCTION / TECHNICAL CHARACTERISTICS</i>	8
<i>INSTALLATION</i>	9
<i>STEAM GENERATOR DESCRIPTION</i>	10
<i>USE INSTRUCTIONS</i>	11
<i>GENERAL CARE AND CLEANING</i>	12

INTRODUCTION :

We would like to congratulate you on purchasing the PRESSOJET steam generator.

This device, which is very simple to use, allows easy removal of coating residues, degreasing of cast alloys and any kind of prosthetic cleaning where steam is required.

IMPORTANT !!!

This machine was designed specifically for use in dental prosthetic laboratories. Any other use will automatically forfeit the manufacturer's guarantee.

TECHNICAL CHARACTERISTICS:

Current and frequency:		230 v 50/60Hz
Power	:	1600 W
Dimensions :		
Height	:	190 mm
Width	:	250 mm
Depth	:	350 mm
Weight	:	12 Kgs
Boiler capacity	:	4 Litres
Boiler using capacity	:	3 Litres
Operation pressure	:	4,5 Bars

INSTALLATION :

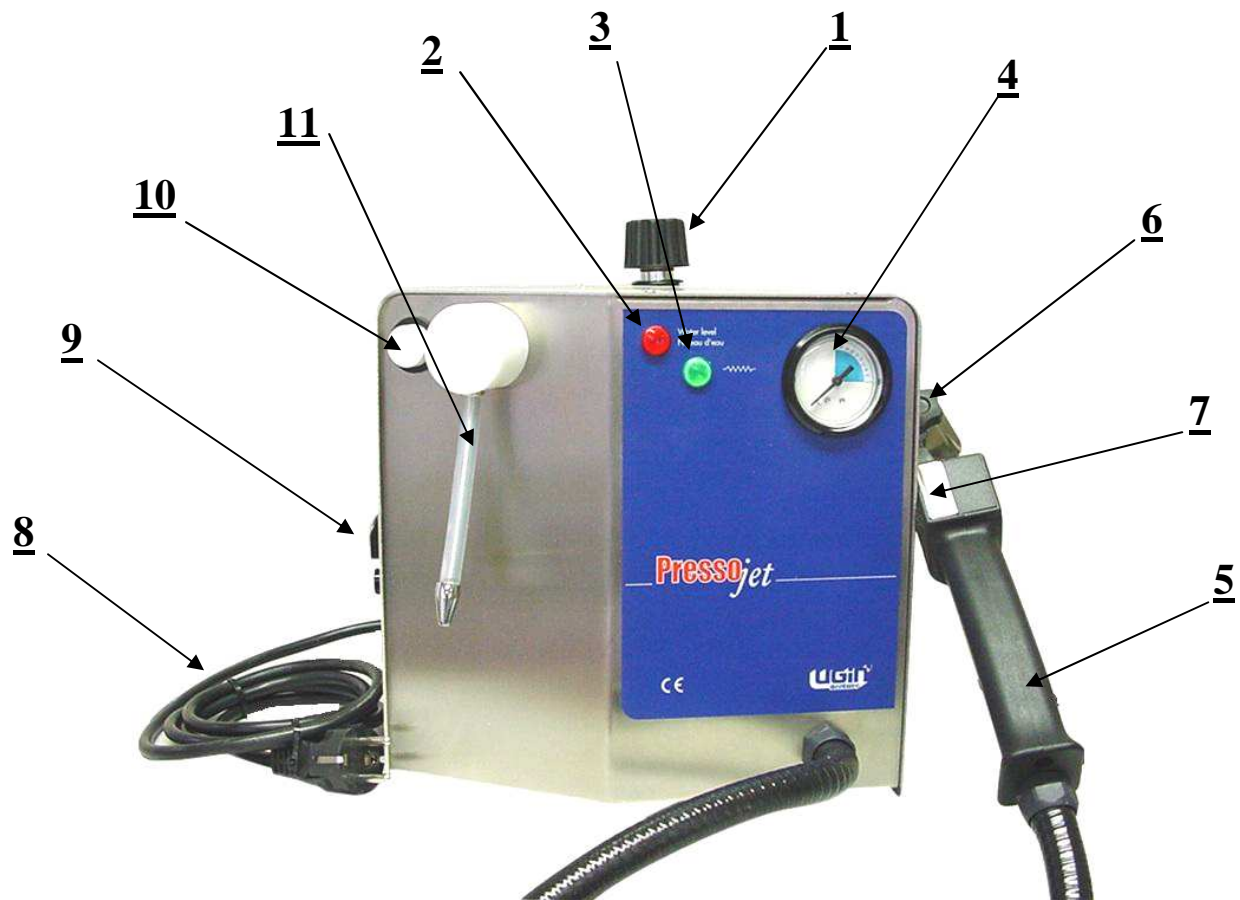
- Make sure that the PRESSOJET is in perfect condition when it is removed from its packing. Notify the carrier of any damage.
- Remember to have the carrier sign the delivery note. Any damage that has occurred should be indicated on the delivery note and notified by recorded delivery letter.

START UP :

- Position the machine on a flat, easily accessible surface.
- Unscrew filler cap **1** located on the upper part of the device.
- Fill the tank with 3 litres of distilled (or demineralised water), using the special container (capacity 2 litres) provided.
- Replace cap **1**.
- Connect the mains lead **8** to a 10/16A earthed socket.

WARNING: To ensure long-term performance of the equipment, it is essential that only distilled (or demineralised) water be used.

STEAM GENERATOR DESCRIPTION :

1: Filler cap

7: Steam gun control

2: Water level indicator

8: Mains supply lead

3: Heat indicator

9: On/Off Switch

4: Pressure gauge

10: Steam nozzle control

5: Gun

11: Steam nozzle

6: Steam flow adjustment

UTILISATION :

■ Once you have filled the tank with 3 litres of distilled (or demineralised) water and closed cap **1** securely, activate switch **9**, located on the left hand side of the device.

■ The heat indicator **3** (green) will come on and will not go out until normal operating pressure is reached (the needle of the pressure indicator **4** moves into the blue area).

WARNING: Never try to unscrew filler cap **1** once the equipment has pressurized (4.5 Bar).

■ Press button **7** on the steam gun **5** to release steam.

■ To obtain steam from the fixed nozzle **11**, press button **10**.

■ When there is no more water in the PRESSOJET tank, the red indicator **2** (water level) will come on and an audible signal confirms that the tank is dry. The device's heater then switches off automatically.

■ It is then necessary to re-fill the machine, following these instructions very carefully:

- Wait for the pressure indicator **4** to return to 0.
- Carefully unscrew filler cap **1**.
- Wait a few minutes longer before starting to re-fill. If water is added too quickly, it could turn to steam in contact with the hot element.

GENERAL CARE AND CLEANING :

■ To ensure long-term performance of this device, it is necessary to clean the water tank using the special additive sent with the PRESSOJET, at least once every three months (or more often under conditions of intense use).

To do this, you must:

- Disconnect the mains supply.
- Mix 5Cl of the product with 75Cl water.
- Remove filler cap **1** then fill the (empty) tank with the 80 Cl prepared solution.
- Turn on the device and allow the machine to warm up for about 15 minutes.
- Using the steam gun (nozzle pointing downwards) release a small amount of steam.
- Disconnect from the mains once more and then leave overnight for the cleaning product to work.
- Unscrew filler cap **1** then remove the cover from the back of the machine (bottom left hand corner).
- Unscrew the tank outlet plug.
- Rinse the tank with running water for 2 to 3 minutes.
- Replace the outlet plug and then the cover.
- The PRESSOJET is now ready for re-filling.

SPECIAL PRECAUTIONS:**Cleaning additive:**

- Never swallow or inhale this product.
- Avoid contact with skin and eyes.
- Never mix with other cleaning products.
- Never leave within reach of children.

14 -19 rue Henri-Dunant 38180 SEYSSINS – France
Tél. National : 04.76.84.45.45 - Fax : 04.76.84.46.46
Tél. International : ++33.4.76.84.45.43 - Fax : ++33.4.76.84.46.46
Email : info@ugin-dentaire.fr - [http : www.ugin-dentaire.fr](http://www.ugin-dentaire.fr)

ME 08/05